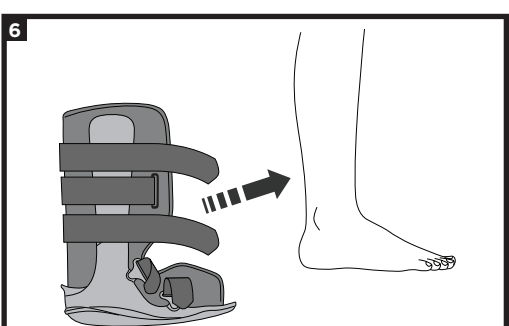
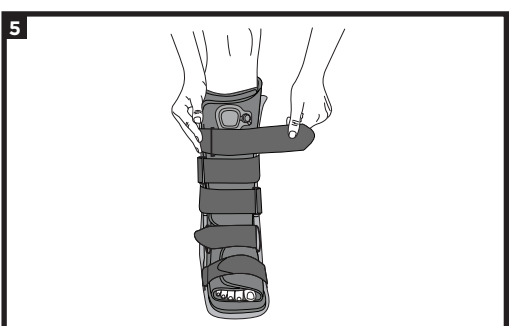
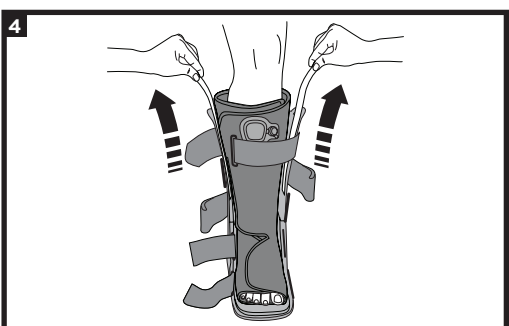
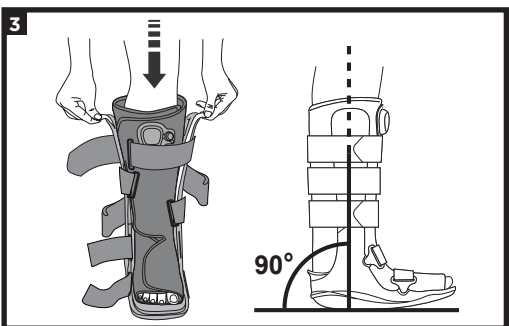
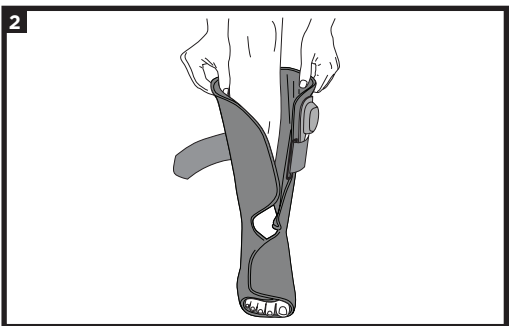
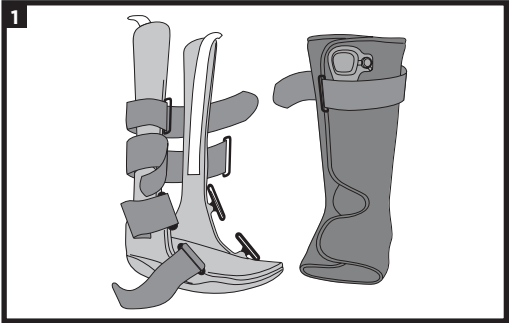
ENGLISH

INDICATIONS

The intended use for Walker, Ankle, and Foot bracing products is to provide support, relief, or protection to the lower leg, ankle, or foot.

INTENDED USERS

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS

- This device is not suitable for unstable fractures or fractures of the tibia or fibula.
- Application of this device is recommended only when the fracture is demonstrably stable and there are acceptable limits of angular and rotational deformity.

⚠ WARNINGS & PRECAUTIONS ⚠

- For single patient use.
- United States Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- The device will not prevent or reduce all injuries.
- Carefully read fitting instructions and warning prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

FITTING INSTRUCTIONS

1. Unfasten hook and loop straps and remove inner soft liner from boot.
2. Open the soft liner and place foot inside with the heel fitting snugly into the back portion of the liner. Fasten and wrap foot flaps then the leg portion of the liner. Ensure liner is snug but not constrictive. Make sure pump is facing outward.
3. Tall boot — Leave strut pull tabs intact. Gently spread the side struts and place the lower leg into the brace. The ankle should be centered at the struts with the foot and ankle at 90°.
4. Tall boot — Once aligned correctly, remove the pull tabs and gently push the struts toward the liner to engage the hook and loop on the walker liner.
5. Fasten the hook and loop straps beginning at the toe then moving up the leg. Do not overtighten. As swelling pattern changes throughout the day due to activities, rest, etc., it may be necessary to adjust the straps for comfort.
6. To remove, simply unfasten the straps without removing them completely from the assembled walker. Unfasten the soft good liner and gently bring the foot and ankle forward and out of the walker. To reapply, place the foot and ankle into the liner and reattach the hook and loop straps (As per Fig. 5).

USE AND CARE

Hand wash the liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.

Wipe plastic parts with damp cloth as needed.

MATERIAL CONTENT

Ethylene-vinyl Acetate (EVA) Foam, Nylon, Polypropylene, Polyurethane, Stainless Steel, Thermoplastic Polyurethane (TPU), Thermoplastic Elastomer (TPE), Urethane

WARRANTY

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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