## ⚠️ WARNINGS ⚠️

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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## Universal Cervical Collar Instructions for Use

## Cuello ortopédico cervical universal

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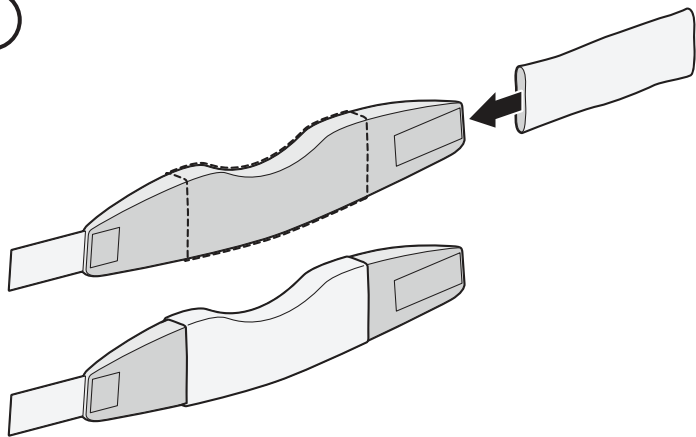
## Collet cervical universel

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Breg, Inc.  
2885 Loker Ave East  
Carlsbad, CA 92010 USA  
P: 1-800-321-0607 | +1-760-795-5440  
F: +1-760-795-5295  
[www.breg.com](http://www.breg.com)

1


Unfasten hook and loop closure and slide Cervical Collar through sockinette. The sockinette should be centered around the dip in the brace.

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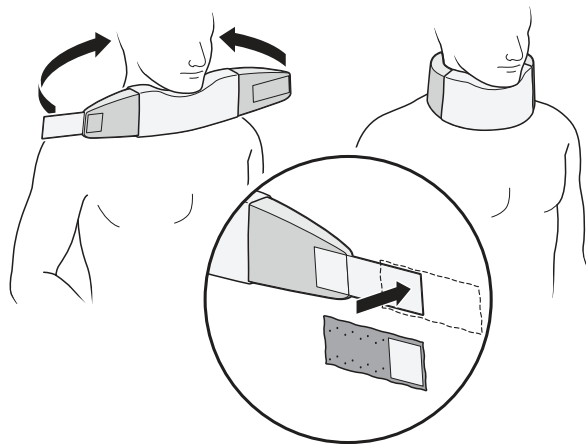2

Rest the patient's chin in dip in Cervical Collar and wrap the collar around the patient's neck. Fasten hook and loop closure. To increase the circumference of the collar, attach blue extension strips to hook and loop closure.

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#### CARE & WASHING INSTRUCTIONS

Remove sockinette from collar. Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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