ASCEND™ ADULT CERVICAL COLLAR 172

MODEL #SP30172-0X0

INSTRUCTIONS FOR USE

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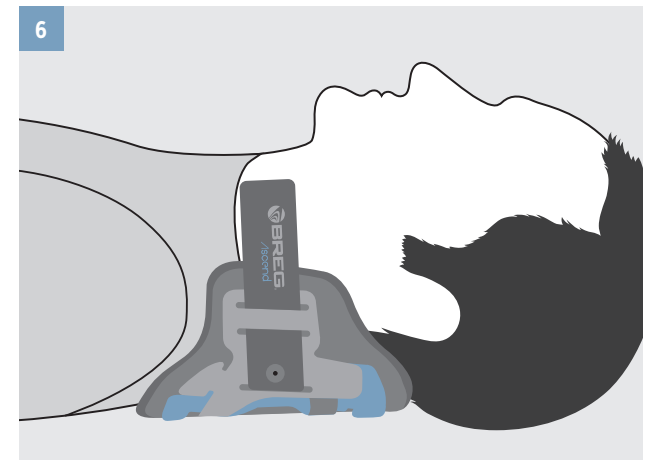
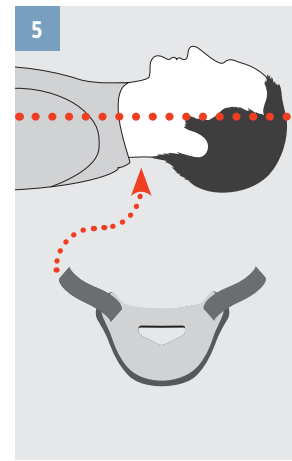
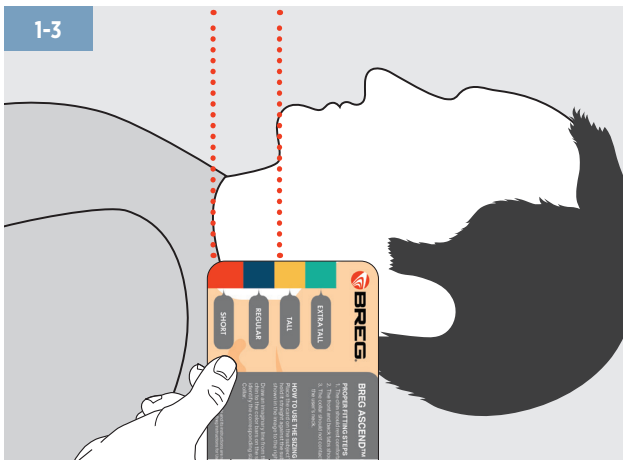
Ascend is a trademark of Breg, Inc.

AW-1.03817 Rev B 07/21

E/U authorized representative
MDSS GmbH, Schiffgraben 41
D-30175 Hannover, Germany

ENGLISH

MEASUREMENT

1. Ensure patient's head is in the desired position.
2. Place the sizing guide against the patient's head with the bottom edge resting on the highest point of the trapezius muscle and flat against the side of the head.
3. Sight an imaginary line from the bottom of the patient's chin to the color bars on the sizing guide to determine the proper size.

BACK PANEL POSITIONING

4. Bend ends of the back panel inward to conform to the shape of the neck.
5. Ensure patient's head is in a neutral position and slide back panel under patient's neck.
6. Ensure loop straps are positioned between patient's ears and shoulders. The back panel should be centered along the midline of the back of the neck.

NOTE: Ensure that the triangular cutout of the back panel is pointed down towards the patient's back.

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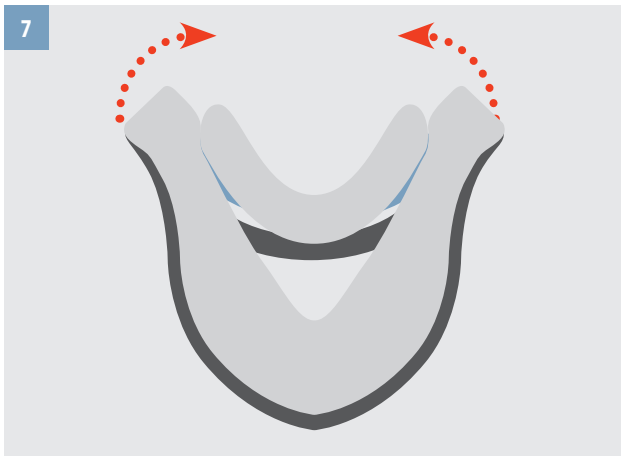
ENGLISH

FRONT COLLAR POSITIONING

7. Bend the ends of the front collar inward to conform to the shape of the neck.
8. Place the bottom of front collar against the patient's chest and align sides with back panel loop straps.
9. Attach straps to the hook on the sides of the front collar.
10. Ensure both straps have an equal amount of tension and proper alignment is maintained, resulting in a snug and supported fit.

NOTE: Ensure that chin piece is not pressing on patient's throat. If it is, adjust side panels of the front collar so they are off the shoulders.

Once fitted, ensure that the height of the front collar is supporting the chin when the patient's head is in a neutral position. If not, confirm sizing is correct.

NOTE: Excess strap material can be trimmed to desired length.

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 WARNINGS 

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the collar, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: This device is offered for sale by or on the order of a licensed health care professional.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

CARE & WASHING INSTRUCTIONS

Hand wash pads in cold water with a mild detergent, rinse and air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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©2021 Breg, Inc. All rights reserved.