PINNACLE CERVICAL COLLAR 180

MODEL #SP40180-000

INSTRUCTIONS FOR USE

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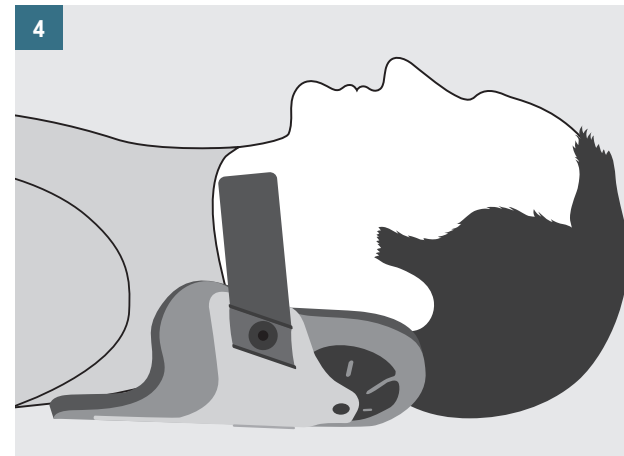
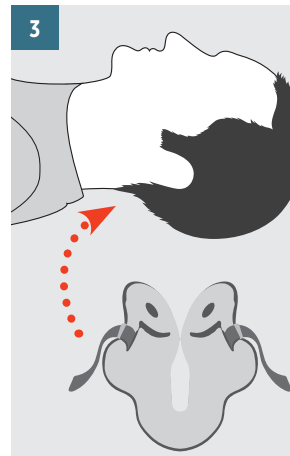
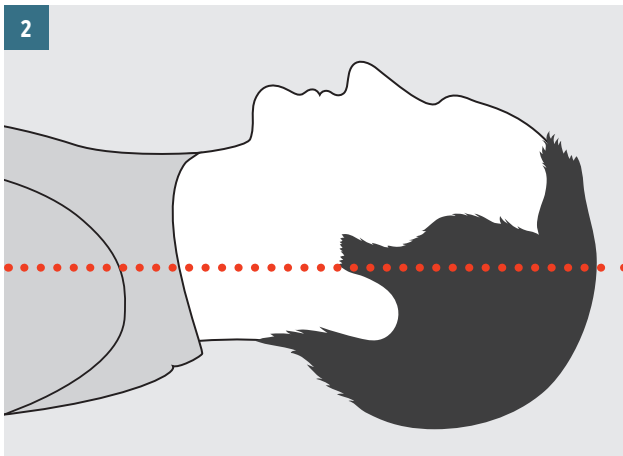
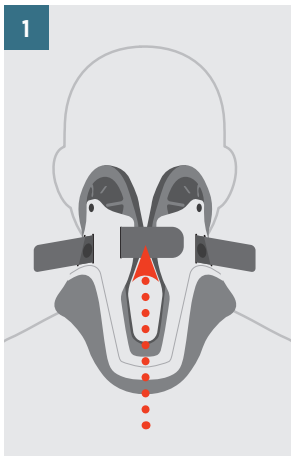
www.breg.com

©2020 Breg, Inc. All rights reserved.
AW-1.03815 Rev B 06/20

E/U authorized representative
MDSS GmbH, Schiffgraben 41
D-30175 Hannover, Germany

ENGLISH

BACK PANEL ADJUSTMENT

1. Prior to placement, adjust the center strap on the back panel to position the multipost discs on either side of the patient's occipital bone.

BACK PANEL POSITIONING

2. Ensure patient's head is in a neutral position.
3. Slide back panel under patient's neck.
4. Ensure loop straps are positioned between patient's ears and shoulders. The back panel should be centered along the midline of the back of the neck.

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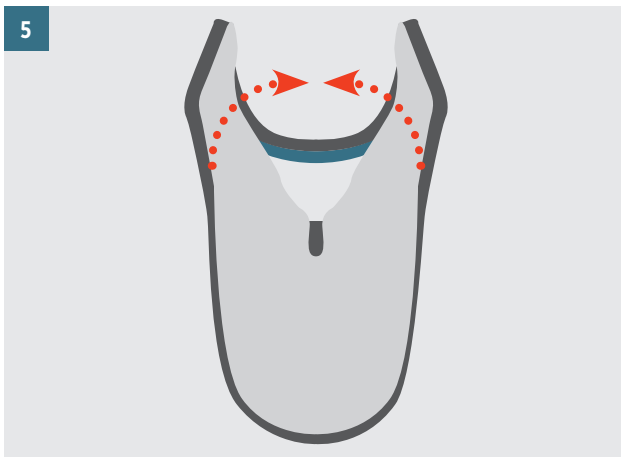
ENGLISH

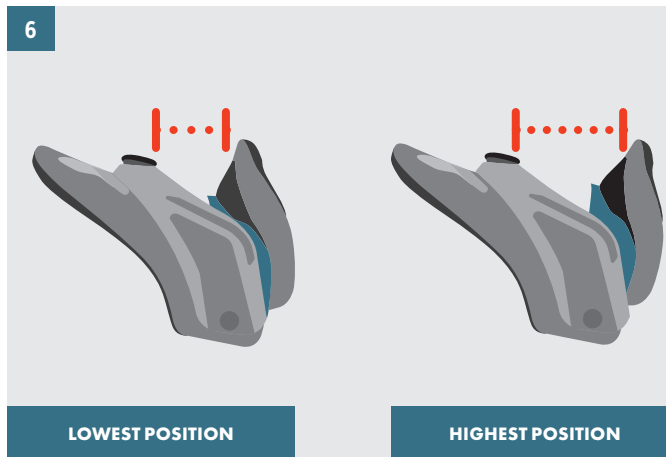
FRONT COLLAR POSITIONING

5. Bend the back ends of the front collar inward to conform to the shape of the neck.
6. The front collar should start with the chin piece in its lowest position.
7. Place the bottom of front collar against the patient's chest and align sides with back panel loop straps.

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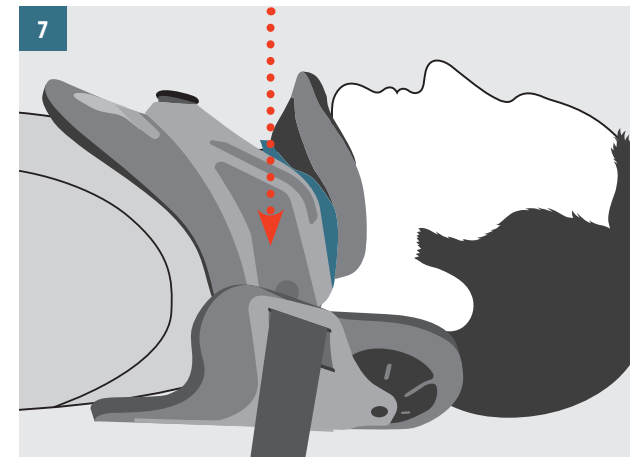
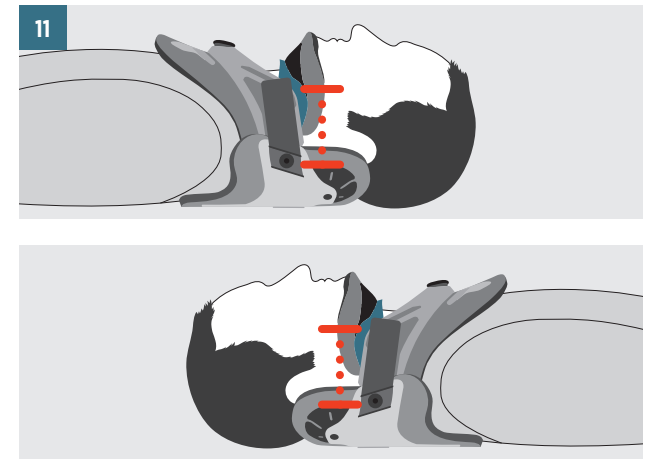
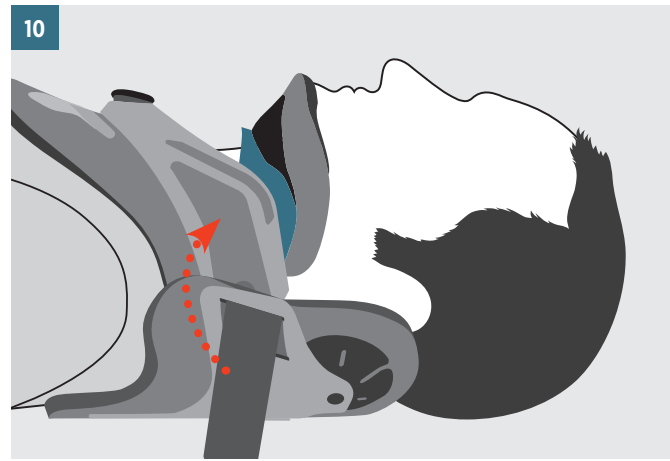
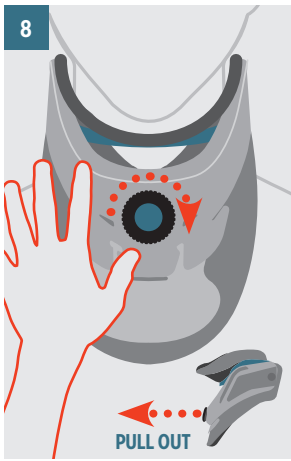
ENGLISH

CHIN ADJUSTMENT

- While holding the front collar in position, **pull out and rotate** the knob clockwise to raise the chin piece until the patient's chin is supported and head is in a neutral position.
- Once adjustments have been made, ensure the knob is secured into one of the stop positions.

STRAP ATTACHMENT

- Attach straps to the hook on the sides of the front collar.
- Ensure both straps have an equal amount of tension and proper alignment is maintained, resulting in a snug and supported fit.

NOTE: Ensure that chin piece is not pressing on patient's throat. If it is, adjust side panels of the front collar so they are off the shoulders.

After application, readjust the back panel center strap as needed to achieve desired positioning of multipost discs.

NOTE: Excess strap material can be trimmed to desired length.

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 WARNINGS 

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the collar, follow all instructions.

WARNING: If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: This device is offered for sale by or on the order of a licensed health care professional.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

CARE & WASHING INSTRUCTIONS

Hand wash pads in cold water with a mild detergent, rinse and air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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