Contender Boxer Splint

Instructions for Use

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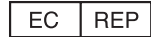
Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com
© 2021 Breg, Inc. All rights reserved.

AW-1.04100 Rev A 01/21

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

Instructions

Left Ulnar Splint Application (Boxer Splint):

Universal sizing and available in Right or Left

Step 1: Apply the brace, seating the palm of the hand into the pre-curved brace.

- Make sure to slide the hand up so the curves in the brace match the mid-hand crease and wrist. Bend as needed so the product forms to the wrist, hand, and fingers.

Step 2: Fasten the middle strap.

Step 3: Fasten the wrist strap.

Step 4: Fasten the finger strap.

Step 5: Brace properly applied.

Care and Cleaning

Hand wash in warm water using a mild detergent. Air dry.

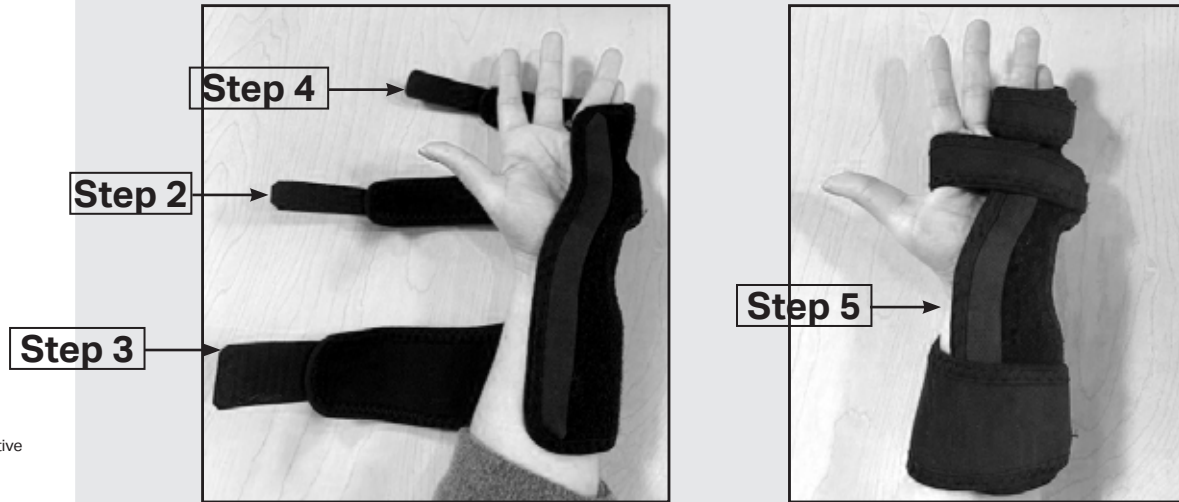
WARNING

CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION. WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION:
IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

IF ANY SERIOUS INCIDENT HAS OCCURRED IN RELATION TO THE DEVICE, REPORT TO BREG AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED.

Left Ulnar Splint Application (Boxer Splint)

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