**PINNACLE
CERVICOTHORACIC
ORTHOSIS 200
UPGRADE**

MODEL #
SP41200-000
SP41204-000

2885 Loker Ave East
Carlsbad, CA 92010 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295





www.breg.com

©2021 Breg, Inc. All rights reserved.

AW-1.04090 Rev A 04/21

E/U authorized representative
MDSS GmbH, Schiffgraben 41
D-30175 Hannover, Germany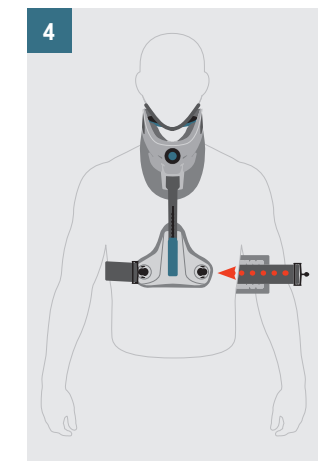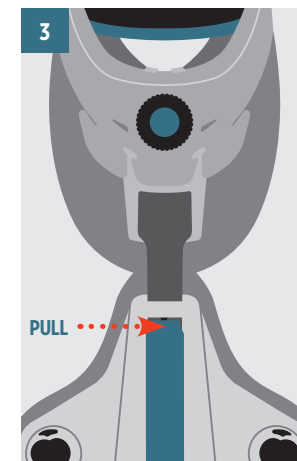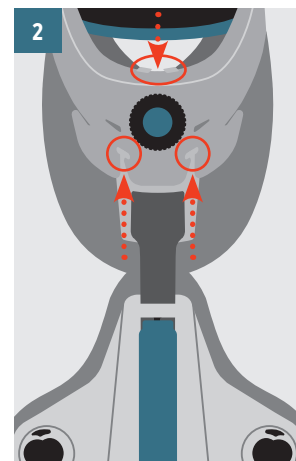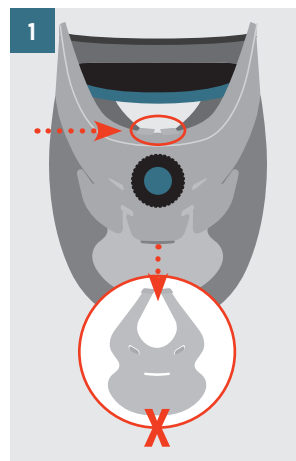

ENGLISH

**PINNACLE CERVICOTHORACIC
ORTHOSIS 200 UPGRADE**

INSTRUCTIONS FOR USE

**CONVERTING A PINNACLE 180
CERVICAL COLLAR TO A CTO**

1. Remove the anterior extension piece from the Pinnacle 180 collar by pushing up on the two hooks located just above the dial.
2. Slide the top of the CTO rigid anterior plate under the dial and ensure the 2 hooks above and below the dial snap into place.
3. Pull blue height adjustment lever continuously while making height adjustment. Slide plate to desired position.
4. Engage thoracic strap and adjust as needed.

Please refer to CTO/CTO4 Instructions for Use for full fitting procedures, warning and cautions.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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
PINNACLE CERVICOTHORACIC ORTHOSIS 200 UPGRADE


MODEL #
SP41200-000
SP41204-000

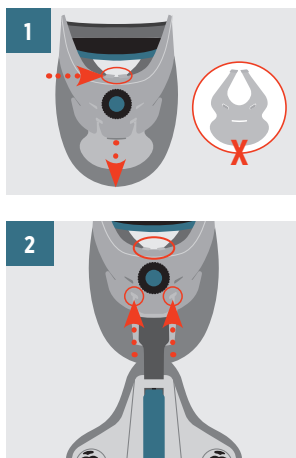
1


2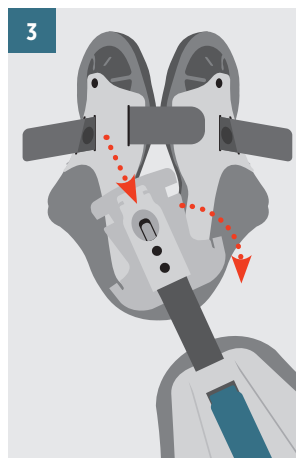


3

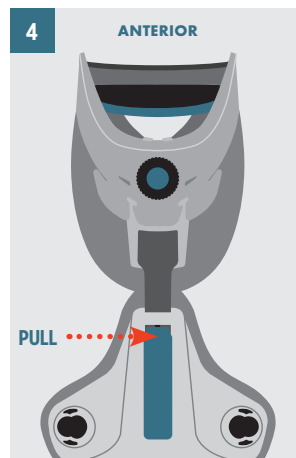4

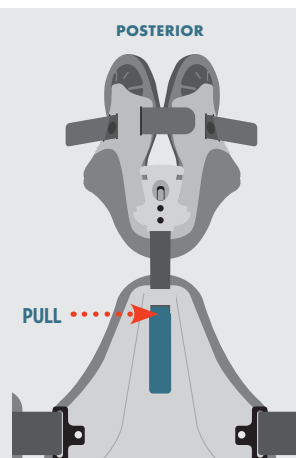5

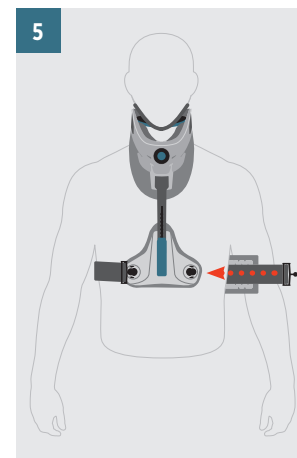6

ENGLISH

PINNACLE CERVICOTHORACIC
ORTHOSIS 200 UPGRADE

INSTRUCTIONS FOR USE

CONVERTING A PINNACLE 180 CERVICAL COLLAR TO A CTO4

1. Remove the anterior thoracic extension piece from the collar by pushing up on the two tabs located just above the dial.
2. Slide the top of the CTO rigid anterior plate under the dial and ensure the 2 tabs above and below the dial snap into place.
3. Slide the top of the posterior plate into the back of the 180 panel. Angle the piece in at the widest point then slide down to snap into place at the bottom of the collar.
4. Adjust the anterior and posterior bars as needed, depending on patient torso length.
5. Adjust the waist belt as needed. Secure waist belt straps to achieve a proper snug fit.

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